

Theragem T1005 Stand Assembly Instructions



The Theragem T1005 Stand is the must-have tool for getting the best out of your Theragem unit.

Compact and sturdy, the stand will safely hold your Theragem unit in place allowing you to effectively target any area of the body delivering a fusion of healing crystal light to your clients.

MEDICAHEALTH





Max Height - 120cm
Weight - 3.2kg

Max Height - 47.2"
Weight - 3.2kg

TO PREPARE YOUR
STAND FOR USE,
FOLLOW THESE
INSTRUCTIONS:

These are the individual stand parts for the Theragem™ T1005.

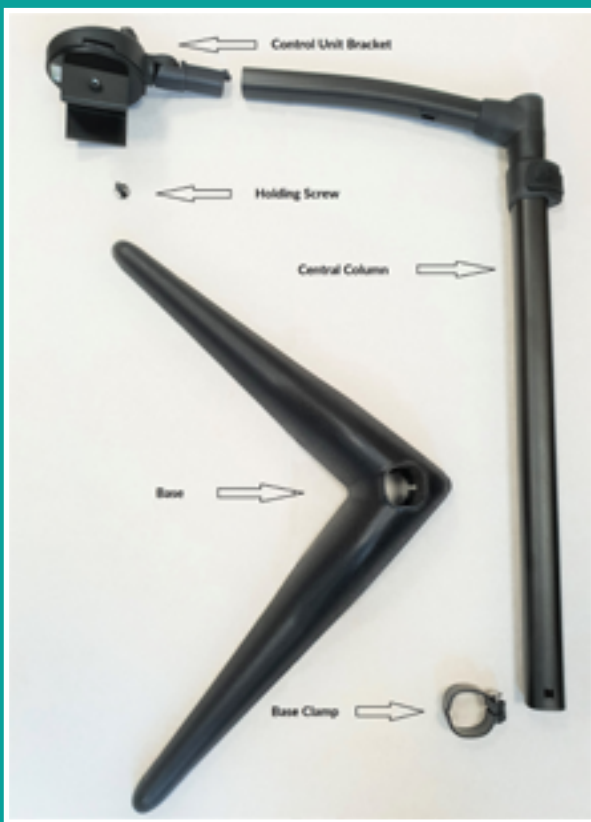

STEP
1

Push the stand's
central column into
the base until it
clicks into place.



STEP
2

Push the base clamp
onto the central
column. Tighten
the screw with a
screwdriver (not
included).

** To minimize the amount of turns
required when fitted to the central
column, before pushing the base clamp
in place, tighten the screw until the base
clamp just barely fits over the central
column.



STEP
3

Push the control unit
bracket into the top
horizontal swing arm, until
the tab clicks into the
receptacle.
**To test this is connected
correctly; pull on the control
unit bracket. Make sure the
bracket does not pull out.



STEP
4



4. Fit the control unit holding screw into place on the side of the control unit bracket, by screwing into place. This will only take a small amount of turning. **If the control unit will not slide into the control unit bracket, unscrew the holding screw slightly. When the control unit is positioned in the bracket, tighten the holding screw to keep the control unit in place during operation.

STEP
5



Adjust the height of your stand by using the quick snap height adjustment lever. Pull the lever outwards to release tension and move the top part of the central column to the required height, push the height adjustment lever inwards until it clicks, to lock it into place. **Always hold the control unit bracket while adjusting the height.

STEP
6

The three tension dials can be tightened or loosened to your requirements with a coin.

**Using a screwdriver will damage the tension dial.



STEP
7

The T1005 Theragem Unit can now be slotted into the holder.



A STAND EXTENSION POLE IS ALSO AVAILABLE TO PURCHASE SEPARATELY

For more information about any of our products you can visit: www.medicahhealth.org

or get in touch directly
+44 (0) 1254 245013 | info@medicahhealth.org

